

INSTRUCTIONS FOR USE

Ambu® Neuroline
Monopolar Neelelektrode
(with pre-attached lead wire)
Single-use

Ambu

Instructions for use Инструкции по применению

Návod k použití

Brugsvejledning

Öffnungsanleitung

Instrucciones de uso

Kasutamisjuhend

Käyttöohje

Mode d'emploi

Upute za upotrebu

Használati útmutató

Istruzioni per l'uso

使用方法

Naujainio instrukcija

Lietotāšanas instrukcija

Gebruiksaanwijzing

Bruksanvisning

Instrukcja użytkowania

Instrucțiuni de utilizare

Istruktör för användning

Инструкции по применению

Návod na použitie

Brusavništvo

Kullanma Talimatı

使用说明

WARNING! Ambu® Neuroline Concentric Needle Electrodes are for single-use only. Reuse on other patients may lead to cross-contamination.

Do not use the needle electrode if the individual is damaged or has been opened to minimize the risk of an infection.

Do not use the needle electrode for electrical stimulation as this may damage the electrode.

Do not use the needle electrode on infected skin as this may lead to tissue infection.

CAUTIONS Ambu® Neuroline Concentric Needle Electrodes are only to be used by healthcare professionals trained in performing Electromyography (EMG) procedures and with the compatible medical materials as indicated may harm the patient.

Do not soak, reuse or sterilize the needle electrodes as these procedures may damage the electrode.

The needle electrodes shall be used with caution on patients who have had previous surgery or who may experience excessive bleeding.

During use of the needle electrode, it should be used as far as possible from the equipment generating electromagnetic interference (EMI) to minimize unwanted radiated electromagnetic signals with artifacts.

It is recommended that this device to be sold by or on the order of a licensed health care practitioner.

1.8. Potential adverse events

Contains Cobalt in stainless steel. Current evidence shows no adverse effects on medical devices with stainless steel covering all populations.

If, during the use of this device or as a result of its use, a serious incident occurs, please report it to the manufacturer and to your national authority.

1.9. General notes

Contains Cobalt in stainless steel. Current evidence shows no adverse effects on medical devices with stainless steel covering all populations.

If, during the use of this device or as a result of its use, a serious incident occurs, please report it to the manufacturer and to your national authority.

NOTE: In case of device malfunction, it is recommended to have additional needle electrodes available.

5. Technical product specifications

5.1. Specifications

Item no. Color Length mm Diameter mm Gauge

74325-36/40 Dark Grey 25 0.35 29 G

74338-36/40 Light Green 25 0.35 29 G

74338-45/40 Dark Green 38 0.45 26 G

74350-36/40 Dark Yellow 50 0.45 26 G

74375-50/40 Dark Red 75 0.50 25 G

Pre-attached lead length: 76 cm.

Duration of use

Duration of use is up to 2 hours.

Storage

Storage temperature

The product shall be stored at 10 to 25 °C (50 to 77 °F).

1. Σημαντικές Απόρρησης – Διάβασε πριν την χρήση

ΕΙΔΟΣ

ΕΛΛΗΝΙΚΑ

